

100% Ethylene Oxide sterilization

ANDERSEN PRODUCTS  
www.ahp.com  
**NOW  
YOU  
CAN**

## The Office Sterilization Solution

*The most popular low temperature sterilization system in North America keeps getting better! Using less than 18 grams of Ethylene Oxide per cycle, Anprolene® gently sterilizes your delicate items that heat and moisture can damage. Our system's unique bag and ampoule method allows for load size flexibility. Easy installation requires only 110V outlet and a dedicated exhaust line to the outside.*



### ▶ Unmatched Sterilization

**Flexibility:** Anprolene's 100% EtO System sterilizes at room temperature without a vacuum or water injection. Protect your most delicate instruments, scopes, drills, fiber optics, cameras, and plastic or rubber items... Literally everything but food, drugs or liquids can be sterilized with Anprolene.

▶ **Sterilization and Aeration in the same cabinet:** The Anprolene system utilizes an advanced purge system that aerates products for two hours at the end of the cycle. Active aeration continues indefinitely until products are removed.

▶ **Unique Load Size Flexibility:** The Anprolene system uses unique plastic sterilization liner bags and individual gas ampoules. We offer two different ampoule/bag combinations to match the size of your sterilization load. You never use more gas than you have to!

▶ **Proven Reliability:** Anprolene has been the choice of healthcare professionals for over 40 years. Anprolene is used in over 5,000 facilities worldwide.

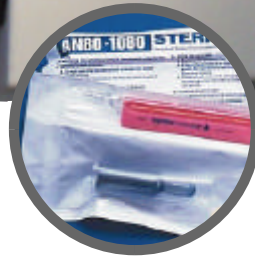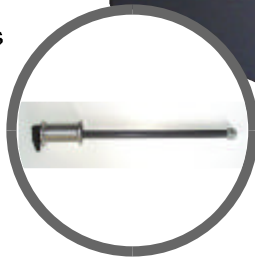

### STARTER KIT INCLUDES

Units	Item	Reorder	Quantity	Retail
25 each	5 cc ampoules and sterilization liner bags	AN71	1 case	\$145.00
20 each	20 cc ampoules and sterilization liner bags	AN79	1 case	\$220.00
50 each	Humidichip relative humidity stabilizer chips	AN1071	2 jars	\$ 20.00
1 each	Humiditube holder	AN1072	1 each	\$ 2.00
50 each	Dosimeter chemical indicators	AN87	2 boxes	\$ 57.40
400 each	EtO exposure indicator strips - chemical indicators	AN85	2 boxes	\$ 60.00
22 each	Steritest biological indicators	AN80	2 boxes	\$100.00
2 rolls	3" Seal & Peel (200 feet per roll)	AN830	1 box	\$ 70.00
1 roll	5" Seal & Peel (200 feet per roll)	AN850	1 box	\$ 55.00
1 roll	7" Seal & Peel (200 feet per roll)	AN870	1 box	\$ 75.00
1 each	Impulse sealer	AN90	1 each	\$130.00
1 each	AirScan test kit (STEL and TWA)	AN93	1 kit	\$ 85.00

**AN2000 Starter Kit**

RETAIL SPECIAL  
~~\$4669.40~~ \$4299.00



the sterility experts

www.sterility.com & www.anpro.com  
Andersen Products, Inc.  
Health Science Park • 3202 Caroline Drive  
Haw River, NC 27258-9564 USA  
800-523-1276 • fax 336-376-8153

www.andcal.com  
H.W. Andersen Products of California, Inc.  
Health Science Park • 3151 Caroline Drive  
Haw River, NC 27258-9789 USA  
800-524-3455 • fax 336-376-3088

Distributed in Europe by:  
H.W. Andersen Products, Ltd.  
Davy Road • Clacton-on-Sea  
Essex CO15 4XA UK  
1255-428-328

Manufactured by:  
Andersen Sterilizers, Inc.  
3154 Caroline Drive  
Haw River, NC 27258 USA


## Sterilizer technical specifications

- ▶ **Anprolene uses less Gas:** The Anprolene system uses less than 18 grams of EtO per cycle. The small quantity of gas used means that the Anprolene system generally meets State and U.S.E.P.A. emission requirements without the need for additional abatement equipment.
- ▶ **Sterility Assurance:** The Anprolene System uses a standard range of sterility assurance indicators to confirm the success of the sterilization cycle. Steritest® is a Biological Indicator (BI), the classic test of sterility. BI's require a minimum 48 hr. culture period. Immediate results are provided by Dosimeter®, a Chemical Indicator, and a rigorous test for every cycle. Anprolene Exposure Indicators are used for each item within a "sterilization load" to ensure individual exposure.
- ▶ **Ease of Installation:** New high pressure, low volume pumps simplify installation, requiring only a 7/8" reinforced PVC exhaust hose to the outside. No water or vacuum line required! Wall mount installation kit included with the sterilizer.


### Specifications: AN2000

Model: AN2000.67  
Exterior dimensions: 22"L x 21.5"H x 18"W  
Inner dimensions: 20"L x 14"H x 14"W  
Total Weight: 50.5 pounds (22.9kg)  
Power Supply: 115 +/-10%, 50/60 Hz/150 Watts  
Manufactured by: Andersen Sterilizers, Inc.  
Distributed by: Andersen Products, Inc.

Outlet Hose  
7/8"OD, 3/4"ID Reinforced PVC Hose

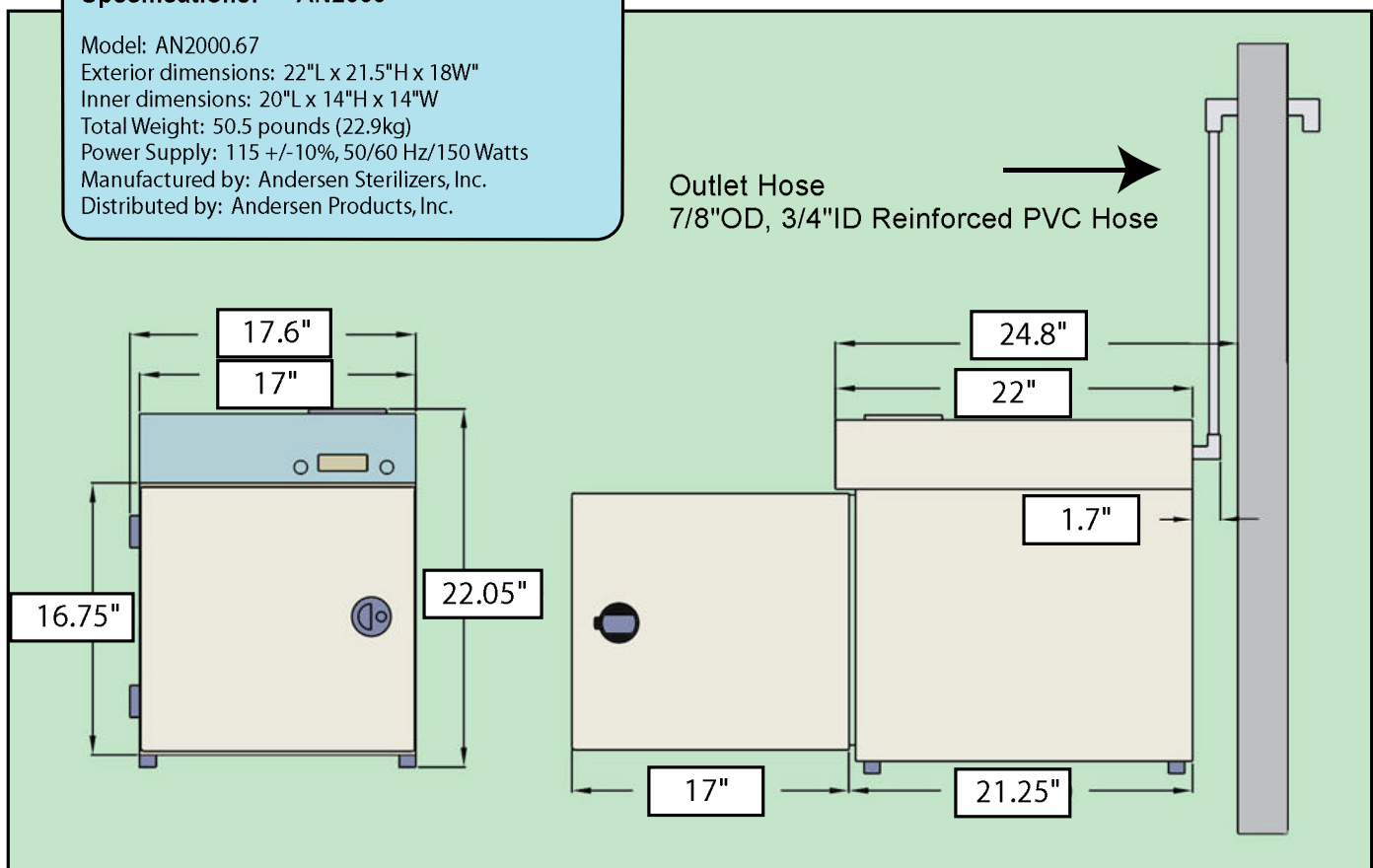

For more information about Anprolene® Sterilization Kits and prices call customer service at 800-523-1276.

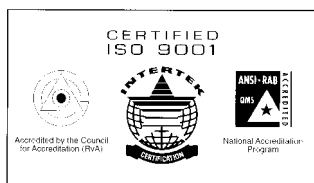

Steritest® is a registered trademarks of Andersen Products, Inc.